# Jamshidi™

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Intended use for the posterior iliac crest biopsy technique.

**Note:** If an aspiration is to be performed, it should be done before the biopsy.

A - Handle

B - Stylet

C - Cannula

D - Probe

E - Probe Guide

F - Support Rod

G - Specimen Cradle

H - Universal Cap

**Warning:** Do not use for a sternal procedure. Due to the needle length, internal thoracic organs or blood vessels may be punctured or otherwise damaged.

#### Aspiration Procedure

- Place the patient in a right or left decubitus position, with the back comfortably flexed and the top knee drawn toward the chest **(1)**. Patient may also be placed in a prone position.
- Locate the posterior superior iliac spine and mark with ink or thumbnail pressure.
- Using sterile technique, prepare the biopsy site.
- Infiltrate the skin, especially the periosteum, with local anesthetic.
- A scalpel blade may be used to puncture the skin prior to inserting the needle.
- Hold the handle securely in the palm of the hand **(2)**.
- Introduce the needle through the incision and, using firm pressure, slowly advance the needle while rotating the needle 45° in an alternating clockwise-counterclockwise motion **(3)**. Entrance into the marrow cavity is generally detected by decreased resistance.
- Rotate the stylet 90° counterclockwise and pull the stylet out of the handle.
- Attach a syringe to the cannula hub **(4)**.
- Apply negative pressure by quickly withdrawing the syringe plunger. Disconnect the syringe and remove the aspirated specimen.

#### Biopsy Procedure

To perform the biopsy after the aspiration, use the same skin incision but choose a different location to enter the bone. For a biopsy only, follow steps 1-8 of the aspiration procedure and continue as follows:

- With stylet removed, slowly advance the needle with a slight clockwise-counterclockwise motion until adequate marrow is obtained. **Note:** Sample architecture may be damaged with excessive clockwise-counterclockwise movement.
- Insert the combined support rod and specimen cradle into the needle **(5)**. Advance only far enough to rest on top of the specimen. Approximation of specimen length can be identified at this time.
- Advance the specimen cradle down to the handle base while maintaining the support rod in a stationary position **(6)**.
- Rotate the specimen cradle 360° **(7)**.
- Remove the specimen cradle and support rod from the needle. Assess adequacy of trephine; or
- Slowly remove the needle, specimen cradle, and support rod from the patient with a gentle clockwise-counterclockwise motion. Remove the specimen cradle and support rod from the needle.
- Advance the support rod through the specimen cradle to remove the biopsy specimen **(8)**; or
- Place the probe guide onto the distal cannula tip. Insert the probe into the guide and remove the biopsy specimen **(9)**.

**Caution:** For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

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Jamshidi, CareFusion and the CareFusion logo are trademarks or registered trademarks of CareFusion Corporation, or one of its affiliates.
© 2010, 2012, 2013, 2014, 2017, CareFusion Corporation or one of its affiliates. All rights reserved.