# Intraosseous Infusion Needle

**en Disposable Intraosseous Infusion Needle**

Do not use if package is damaged.

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Do not use if  
package is damaged.

Carefusion and the Carefusion logo are trademarks or registered trademarks of CareFusion Corporation, or one of its affiliates.

© 2010, 2011, 2013, 2014, 2017, CareFusion Corporation or one of its affiliates. All rights reserved.

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




Made in Dominican Republic by  
**CareFusion DR 203 LTD**

Sponsored by  
**CareFusion Australia 316 Pty. Ltd.**  
**CareFusion New Zealand 313 Ltd.**

362-23305 • 2017-02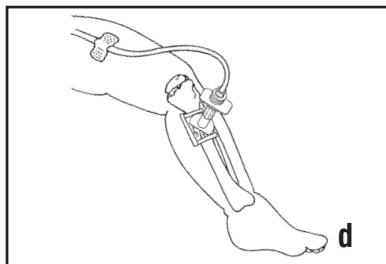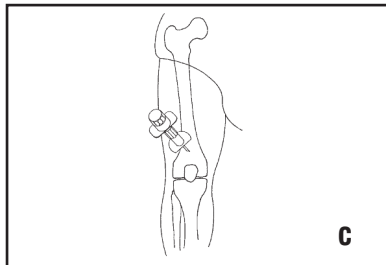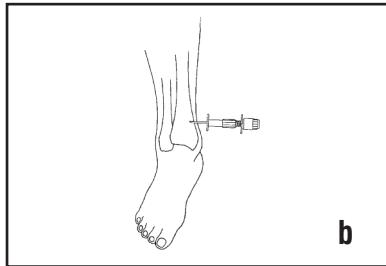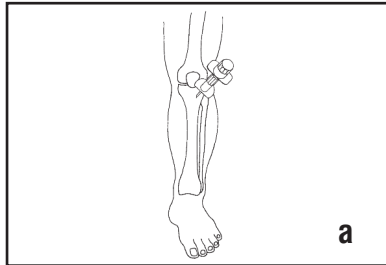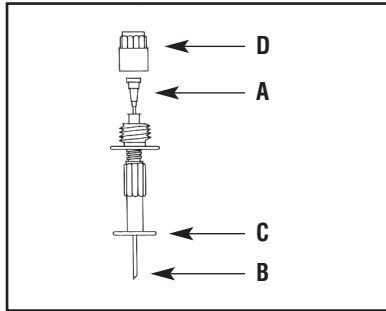

The Disposable Intraosseous Infusion Needle provides for a safe and simple technique to gain vascular access to deliver fluids and drugs to critically ill or injured children. Intraosseous infusion is recommended only in life-threatening emergencies where vascular access is critical and conventional forms of access are not available or are unobtainable.

#### Indications

Fluids administered by intraosseous infusion can be warranted as a clinical alternative in infants and children in such clinical circumstances as cardiac arrest, shock, trauma, or whenever the potential benefits of rapid venous access outweigh the low incidence of complications.

- A. Styler
- B. Cannula
- C. Depth Guard
- D. Cap

#### Contraindications

The procedure may not be used under the following circumstances:

- osteogenesis imperfecta
- osteoporosis
- septic shock/severe systemic infections
- areas of cellulitis
- infected burns
- femur or tibia fracture as fluid may extravasate into subcutaneous tissue

**Note:** Intraosseous infusion should be used only until venous access is obtained and is rarely needed after resuscitation and stabilization (1-2 hours). Intraosseous infusion should be performed with local anesthesia on anyone who is awake and alert.

#### Recommended Needle Placement

##### #1. Proximal Tibia

Use the flat, medial surface of the proximal tibial shaft (tibial plateau) 1-2 cm below the tibial tuberosity (1-2 fingers width). The proximal tibia site may be used in children up to the age of 5-6 years (a).

##### #2. Distal Tibia

Use the medial surface of the tibia, proximal to the medial malleolus where the cortex of the bone and the overlying tissue are both thin. The distal tibia is a preferred site in older children (b).

##### #3. Distal Femur

Use the lower third of the femur in the midline, approximately 3 cm above the lateral condyle. Because of the overlying muscles and fat, it is sometimes difficult to palpate bony landmarks (c).

#### Intraosseous Technique

1. Secure the leg by placing a blanket behind the knee. Prepare the selected site using aseptic technique with povidone iodine solution.
2. Use local anesthesia for patients who are awake. An anesthetic is not required for patients with a depressed mental status such as cardiac arrest.
3. Place the needle to the bone at a 90° angle directed away from the joint space and apply downward pressure with a boring and twisting motion to advance the needle. The needle has an adjustable guard which controls the depth of entry from the selected infusion site. If greater depth of insertion is required, remove the depth guard. **Caution:** To prevent resistance when removing the styler from the cannula, avoid over tightening the depth guard to the upright position.
4. Apply pressure until reaching the bone marrow, which is indicated by a decrease in resistance. The distance from the skin through the cortex is rarely more than 1 cm in an infant or child. The needle should stand without support.

**Note:** To prevent advancing the needle through the opposite side of the bone, hold the needle in the palm of the hand with the index finger 1 cm above the bevel of the needle to avoid pushing it past this mark.

5. Unscrew the cap and remove the styler. Connect a saline filled 5 cc or 10 cc syringe to the needle. Aspirate marrow into the syringe, infuse saline to clear the needle and confirm proper placement.

**Note:** Marrow may not always draw into the syringe in spite of correct placement.

6. Connect the needle to conventional intravenous infusion tubing using gravity or pressure to start infusion.
7. The needle should be flush with the skin and can be taped in place. Regulate fluids to assure proper infusion. Rapid infusions during shock states may not be effective by gravity alone and may require pressure infusion (d).
8. Observe the site for extravasation of fluid, an indication that the placement is too superficial or that the bone has been pierced through both sides.

#### See reverse side for Disposable Illinois Sternal/Iliac Bone Marrow Aspiration Needle Instructions for Use.

**Caution:** For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

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



Do not use if  
package is damaged.  
如包装破损,请勿使用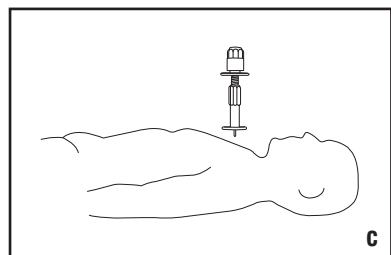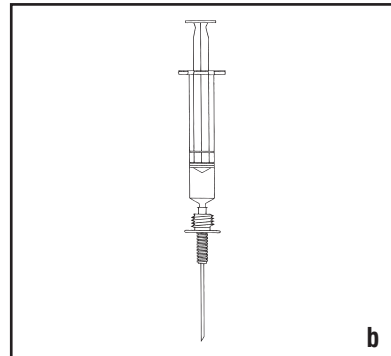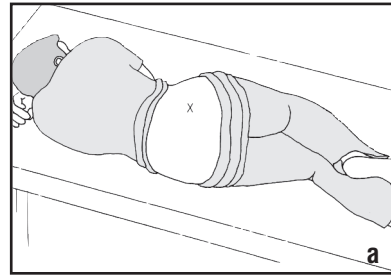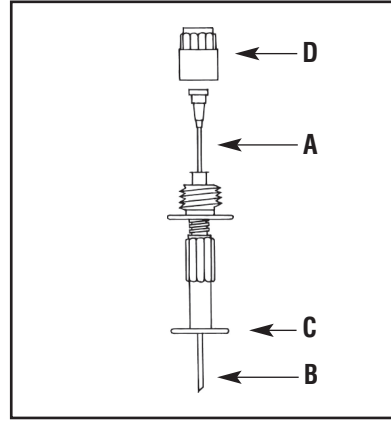

Intended use to aspirate marrow from either the sternum or the iliac crest.

- A. Stylet
- B. Cannula
- C. Depth Guard
- D. Cap

#### Iliac Crest Aspiration Technique

- Place the patient in a right or left decubitus position, with the back comfortably flexed and the top knee drawn toward the chest (a). Patient may also be placed in a prone position.
- Locate the posterior superior iliac spine and mark with ink or thumbnail pressure.
- Using sterile technique, prepare the puncture site.
- Infiltrate the skin, especially the periosteum, with local anesthetic.
- A scalpel blade may be used to puncture the skin prior to inserting the needle.
- Remove the depth guard from the needle. Introduce the needle through the incision and, using firm pressure, slowly advance the needle while rotating the needle 45° in an alternating clockwise-counterclockwise motion. Entrance into the marrow cavity is generally detected by decreased resistance.
- To remove the stylet, unscrew the cap. Pull the stylet out of the cannula.
- Attach a syringe to the cannula hub (b).
- Apply negative pressure by quickly withdrawing the syringe plunger. Disconnect the syringe and remove the aspirated specimen.

#### Sternal Aspiration Technique

**Note:** The depth guard should be used for sternal procedures to help control depth of entry.

- Place the patient in a supine position (c).
- Using sterile technique, prepare the puncture site.
- Infiltrate the skin, especially the periosteum, with local anesthetic.
- A scalpel blade may be used to puncture the skin prior to inserting the needle.
- Adjust the depth guard to the desired position. **Caution:** To prevent resistance when removing the stylet from the cannula, avoid over tightening the depth guard to the upright position. Introduce the needle through the incision and, using firm pressure, slowly advance the needle while rotating the needle 45° in an alternating clockwise-counterclockwise motion. Entrance into the marrow cavity is generally detected by decreased resistance.
- To remove the stylet, unscrew the cap. Pull the stylet out of the cannula.
- Attach a syringe to the cannula hub.
- Apply negative pressure by quickly withdrawing the syringe plunger. Disconnect the syringe and remove the aspirated specimen.

**Note:** It is advisable to obtain a bone marrow aspirate in conjunction with a core biopsy specimen, since aspiration alone does not always permit an adequate evaluation of the functional state of the marrow.

**See reverse side for Disposable Intraosseous Infusion Needle Instructions for Use.**

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