Jamshidi, Carefusion and the Carefusion logo are trademarks or registered trademarks of CareFusion Corporation, or one of its affiliates.

© 2010, 2011, 2013, 2014, 2017, CareFusion Corporation or one of its affiliates. All rights reserved.

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Made in Dominican Republic by  
CareFusion DR 203 LTD

Sponsored by  
CareFusion Australia 316 Pty. Ltd.  
CareFusion New Zealand 313 Ltd.

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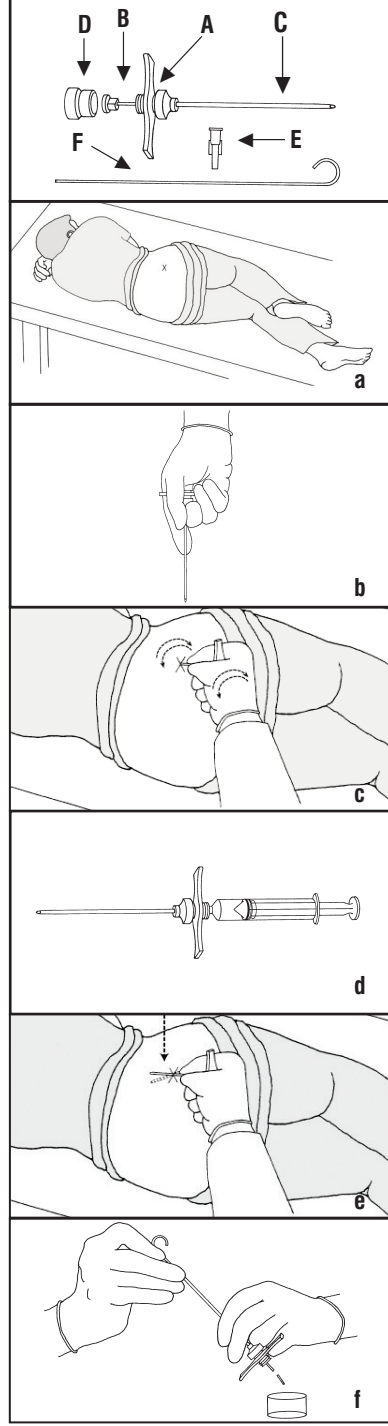
**PMS Black**

DCN 60711  
 Vernon Hills, IL  
 David Knuth  
 02-08-17  
 362-23306

Proofed by: \_\_\_\_\_ Date: \_\_\_\_\_  
 Dimensions checked: \_\_\_\_\_ Copy checked: \_\_\_\_\_

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en

Intended use for the posterior iliac crest biopsy technique.

**Note:** If an aspiration is to be performed, it should be done before the biopsy.

- A - Handle            D - Cap
- B - Stylet            E - Adapter
- C - Cannula          F - Probe

**Warning:** Do not use for a sternal procedure. Due to the needle length, internal thoracic organs or blood vessels may be punctured or otherwise damaged.

**Aspiration Procedure**

1. Place the patient in a right or left decubitus position, with the back comfortably flexed and the top knee drawn toward the chest (a). Patient may also be placed in a prone position.
2. Locate the posterior superior iliac spine and mark with ink or thumbnail pressure.
3. Using sterile technique, prepare the biopsy site.
4. Infiltrate the skin, especially the periosteum, with local anesthetic.
5. A scalpel blade may be used to puncture the skin prior to inserting the needle.
6. Hold the handle securely in the palm of the hand (b).
7. Introduce the needle through the incision and, using firm pressure, slowly advance the needle while rotating the needle 45° in an alternating clockwise-counterclockwise motion (c). Entrance into the marrow cavity is generally detected by decreased resistance.
8. Unscrew the cap and pull the stylet out of the handle.
9. Attach a Luer slip syringe to the cannula hub. Use the adapter for a Luer lock syringe (d).
10. Apply negative pressure by quickly withdrawing the syringe plunger. Disconnect the syringe and remove the aspirated specimen.

**Biopsy Procedure**

To perform the biopsy after the aspiration, use the same skin incision but choose a different location to enter the bone. For a biopsy only, follow steps 1-8 of the aspiration procedure and continue as follows:

11. With stylet removed, slowly advance the needle with a slight clockwise-counterclockwise motion until adequate marrow is obtained. **Note:** Sample architecture may be damaged with excessive clockwise-counterclockwise movement.
12. With a quick motion, completely rotate the needle 360° ten times to the right and ten times to the left while applying light downward pressure to allow the tapered tip to help sever the biopsy specimen.
13. Pull the needle back two or three millimeters and, with minimal pressure, direct its tip at a slightly different angle. Advance the needle two or three millimeters further, and rotate 360° two more times. This procedure further helps to sever the specimen before withdrawing the needle (e).

**Caution:** Applying too much pressure when redirecting the needle may bend the needle. Redirecting the needle beyond 20° may cause the needle to bend or break.

14. Slowly remove the needle with the same clockwise-counterclockwise motion.
15. Insert the probe into the distal cannula tip to remove the biopsy specimen (f).

**Caution:** For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

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